


Proclear® EP

Performs like a multifocal,
fits like a sphere

Proclear® EP is designed specifically for the visual needs of the early presbyope. The lens has a centre spherical distance zone with a progressive aspheric zone that provides excellent intermediate and near vision.

- As simple to fit as a sphere.
- Especially designed for patients with up to +1.25 Add.
- Perfect way to introduce patients to multifocal contact lenses; making the transition into Balanced Progressive™ Technology easier as their condition progresses.
- Utilises the unique PC Technology.

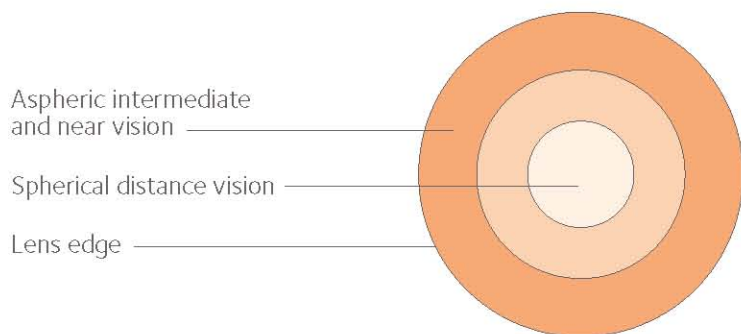

Specification	Proclear® EP
Mater content	60%
Manufacturing process	Fully cast moulded
Base curve	8.7mm
Diameter	14.4mm
Power range (0.50 steps after -6.50DS)	+6.00 to -8.00DS
Add	Suitable for up to a +1.25 Add
Centre thickness (@-3.00DS)	0.16mm
Dk (ISO 9913-1 – 1998)	25
Dk/t (@-3.00DS)	16
Wear indications	Daily wear
Handling tint	
Light blue	
Pack size	3 pack monthly disposable format
Trial lens	Single lenses available in all parameters

Fitting procedure.

- Fit using the spectacle Rx.
- Vertex sphere if over +/-4.00D.
- No need to determine eye dominance.
- No need to compensate for Add (assuming patient's Add is not above +1.25D).
- Leave the lights on in the exam room to eliminate pupil dilation.
- Use loose trial lenses for the over refraction (+/-0.25D), do not use a phoropter.

XSMXXXX