



**Ironically, we offer a
3-year warranty.
On a 15-second job.**



With its unrivaled 3-year warranty and also a 15-second purge and lubrication cycle, you can count on SciCan's STATMATIC™ handpiece maintenance system to properly maintain your handpieces for a long time to come. Increase the life and performance of your handpieces and chucks by ensuring thorough purging and lubricating.

The STATMATIC efficiently services 3 handpieces (45 second total runtime) and does it all mess free with no lubricant spills or leaks.

For more information on the time saving STATMATIC visit www.scican.com.

Your Infection Control Specialist™

SciCan Dental

The ultimate care system for 3 handpieces and their chuck mechanisms.



STATMATIC is the automatic care and maintenance system for up to 3 instruments of your choice, plus their chuck mechanisms. STATMATIC optimizes working procedures in your practice and plays an important part in protecting and maintaining your valuable instruments.

ONE, TWO OR THREE. STATMATIC WORKS IT OUT.



QUICK AND THOROUGH:

- Automatically purges and maintains up to three handpieces.
- Cycle time is only 15 seconds per handpiece.
- Unique chuck care system.
- Optimal lubricant coverage throughout the handpiece.
- Compressed-air connection: Excess lubricant purging.
- Automated reproducible quality of care.
- Easy spray can insertion.
- Can coupling with unique safety valve.
- Absorption pad for excess lubricant.
- 3 years guarantee – only from SciCan.

CONCLUSION:

Improved working life of your handpieces through efficient purging and lubrication – saving you time and money.




CHANGING THE CAN

It's really as easy as can be: push up the retaining ring, snap the spray can into place and let the retaining ring back down again.



CHANGING THE ABSORPTION PAD

Clean as a whistle: STATMATIC is fitted with an absorption pad to retain any excess lubricant.

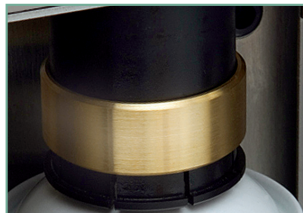

SAFETY VALVE

Absolutely safe: the automatic safety valve will only open when a care programme has been activated.



CHUCK CARE: CM CARE

Put the chuck in position, press the button, done!



TECHNICAL DATA OF STATMATIC

Dimensions (W x D x H):	Weight:	Electrical ratings:	Protection class:	Ambient temperature:	Sound level:	Relative humidity:	Compressed air:	Air consumption:	Compressed-air quality:
285 x 190 x 400 mm (11.4 x 7.6 x 16 inches)	8.6 kg 18.9 lbs	Rated voltage: 100-230 V Rated frequency: 50/60 Hz Rated power: max. 30 VA	1	5-40° C (41-104 °F)	60 (65) dB (A)	max. 85 %	4-6 bar (58-87 psi)	40 NL/min	Residual oil content: max. 0.1 mg/m ³ Residual dust: max 1µ; max. 1 mg/m ³ Residual water: max. 0.1 g/m ³ at -40 °C



SciCan Ltd. 1440 Don Mills Road, Toronto, Ontario
Canada M3B 3P9
Tel (416) 445-1600 Fax (416) 445-2727
International fax (416) 446-2734



3 YEAR GUARANTEE:

Highest quality stainless steel design, long working life – you can rely on STATMATIC. Quality – Made in Germany.

CANADA: SciCan Ltd. 1440 Don Mills Road, Toronto, Ontario M3B 3P9 / Phone (416) 445-1600 Fax (416) 445-2727 Toll free 1-800-667-7733 custservice.ca@scican.com

GERMANY: EU Representative: SciCan GmbH, Wangener Strasse 78, 88299 Leutkirch, Germany, Tel.: +49 (0)7561 98343 - 0 Fax: +49 (0)7561 98343 - 699

SWITZERLAND: SciCan Medtech AG, Alpenstrasse 16, 6300 ZUG, Switzerland, Tel: +41 (0) 41 727 7027 Fax: +41 (0) 41 727 7029

USA: SciCan, Inc. 701 Technology Drive, Canonsburg PA, 15317 / Phone (724) 820-1600 Fax (724) 820-1479 Toll free 1-800 572-1211 uscustomerservice@scican.com

SciCan Dental

Your Infection Control Specialist™